


# INTERTON APP - USER GUIDE

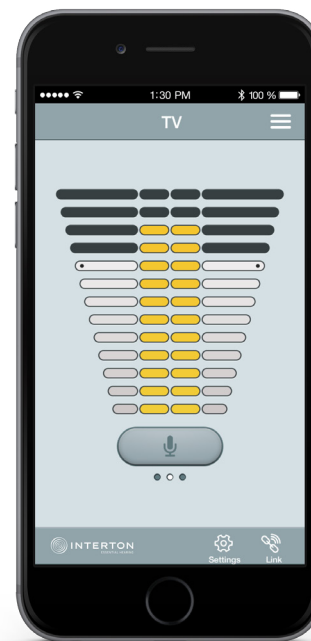

# Introduction

**Congratulations on downloading your Interton app. The Interton app is made to enhance your hearing experience and enable you to get more out of your Interton hearing aids.**

Interton's innovative sound technology and design, combined with customized device programming by your hearing care professional, will make hearing a more enjoyable experience.

The Interton app enables you to get more out of your Interton hearing aids by providing new ways to gain control.

For more information and help for apps, please visit the website:

**[www.interton.com/support](http://www.interton.com/support)**


If you would like a printed version of the app user guide please contact customer support or simply print this user guide.



# Finding and installing the app




## Access apps for iPhone, iPad and iPod touch – download on the Apple App Store by following these steps:

1. Enter the App Store by tapping the App Store icon  on an iPhone, iPad or iPod touch
2. In the App Store, search for “Interton” and the name of the app
3. If using an iPad, change the search criteria in the top left corner to “iPhone only”
4. After finding the app, tap “Get”
5. Then tap “Install”
6. Enter a valid Apple ID and password
7. After entering the password, the Interton app will download and install
8. After download, you can find the app on the mobile device screen - tap to open the Interton EasyHearing app



## Access apps for Android devices – download at the Google Play Store by following these steps:

1. Enter the Play Store by tapping the Play Store icon  on your Android phone.
2. In the Play Store, search for “Interton” and the name of the app
3. After finding the app, tap “Install” then “Accept” to allow the app to access certain features on your phone such as Bluetooth
4. After entering the password, the Interton EasyHearing app will download and install
5. When installation is complete, tap “Open” to open the Interton EasyHearing app

# Interton EasyHearing app




The Interton EasyHearing app provides control of your hearing aids. Used with the PhoneClip 2, Interton EasyHearing lets you use your mobile device to easily obtain an overview of and to remotely control your Interton wireless hearing aids and Interton wireless accessories.

## To use the app, you need the following devices:

1. Wireless accessory: Interton PhoneClip 2
2. Hearing aid: Interton wireless-enabled hearing aids
3. Mobile device:
  - iPhone, iPad or iPod touch running iOS 7 or later
  - Android<sup>1</sup> smartphone running OS version 2.3.5 or later


For best performance, the Interton EasyHearing app should not be used actively on more than one mobile device at a time.

For more information, please consult **[www.interton.com/support](http://www.interton.com/support)**

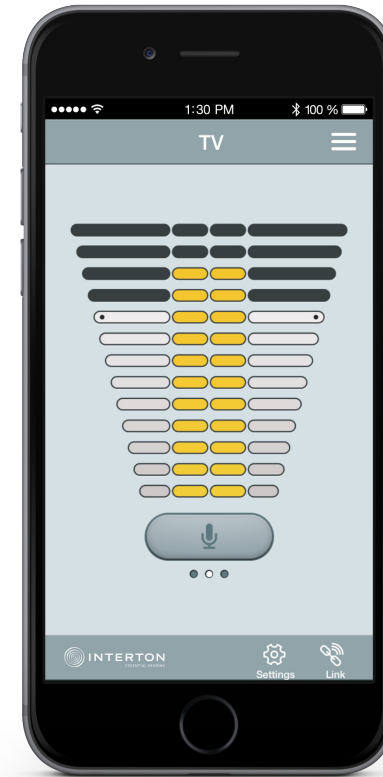

<sup>1</sup>Functionality verified on selected models from HTC, Huawei, LG, Motorola, Samsung and Sony.


# Interton EasyHearing app functionality

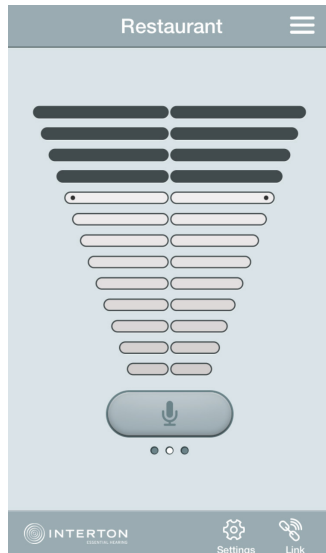

Adjust or mute volume level on your wireless hearing aids

Adjust or mute volume level on your wireless accessories

Use the dedicated link screen to overview connections between the app and your hearing aids

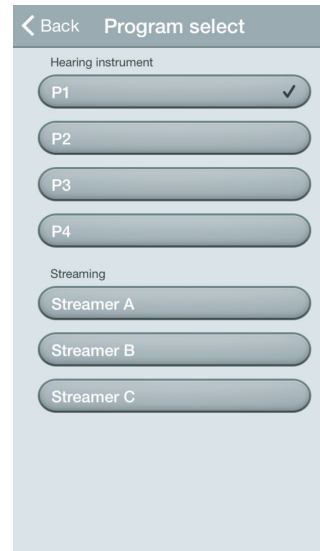

Change manual and streamer programs

## Intended use of smart phone apps:

The apps are intended for use with the hearing aids to control hearing-aid functionality and to provide the end user with additional options for adjusting hearing parameters as well as to provide extra, relevant features.

Interton mobile-device-connected apps are intended for use with Interton wireless hearing aids. Interton mobile device apps send and receive signals from the Interton wireless hearing aids via mobile devices for which the apps have been developed.



GN ReSound A/S  
Lautrupbjerg 7  
DK-2750 Ballerup  
Denmark

Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S

# Warnings and precautions


## **Use with mobile device apps:**

Notifications of app updates should not be disabled, and it is recommended that the user installs all updates to ensure that the app will function correctly and will be kept up to date.

The app must only be used with Interton devices for which it is intended, and Interton takes no responsibility if the app is used with other devices.

## **Warnings and cautions**

If using the Interton app with the hearing aid, please make sure to read the hearing aid user guide. The hearing aid user guide was included in the hearing aid package. If you need a fresh copy, please consult Interton customer support.

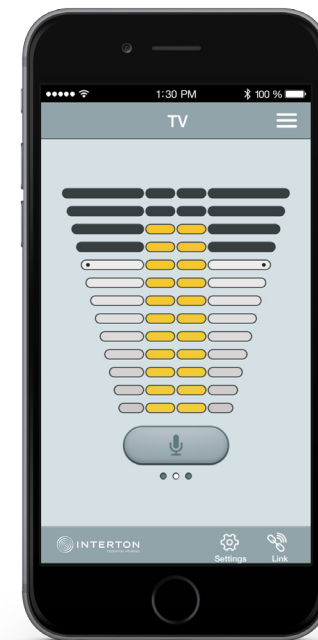



### **Interton - Essential Hearing**

Founded in Germany in 1962, Interton today is part of one of the world's largest hearing instrument organisations, the GN Group. We use their proven core technology in a way that falls in line with our own way of doing things. We believe that most people need a straightforward, functional and easy-to-use hearing solution at a fair price. So that's what we give them.

**Interton A/S** | Lautrupbjerg 7 | DK-2750 Ballerup

Tel: +45 4575 1111 | [www.interton.com](http://www.interton.com)

© The GN ReSound Group, 2015. Apple, iPhone, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android and Google Play are registered trademarks of Google Inc. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. Samsung, Galaxy and Galaxy S5 are trademarks of Samsung Electronics.