




App user guide

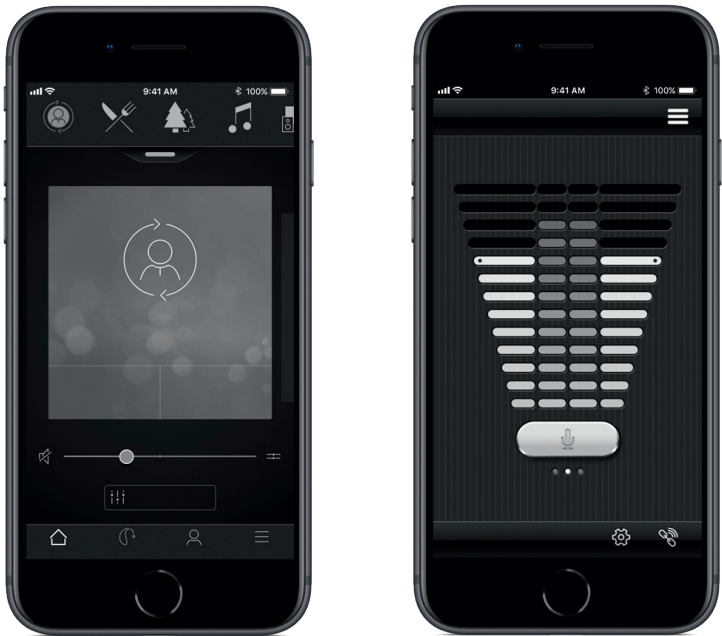

Interton.com

Introduction

The Interton apps are made to enhance your hearing experience and enable you to get more out of your Interton hearing aids.

Interton's innovative sound technology and design come in a full line-up of hearing aids that make hearing a more enjoyable experience. The hearing aids are available exclusively through audiologists and hearing care professionals.

The Interton apps enable you to get more out of your Interton hearing aids by providing new ways to gain control, personalization and relief.

For more information and help for apps, please visit the website:
www.interton.com


If you would like a printed version of the app user guide please contact customer support or simply print this user guide.



Finding and installing the app




Access apps for iPhone, iPad and iPod touch - download on the App Store by following these steps:

1. Enter the App Store by tapping the App Store icon  on an iPhone, iPad or iPod touch
2. On the App Store, search for "Interton" and the name of the app
3. If using an iPad, change the search criteria in the top left corner to "iPhone only"
4. After finding the app, tap "Get"
5. Then tap "Install"
6. Enter a valid Apple ID and password
7. After entering the password, the Interton app will download and install
8. After download, you can find the app on the mobile device screen - tap to open the Interton app



Access apps for Android devices - download on Google Play by following these steps:

1. Enter Google Play by tapping the Play Store icon  on your Android phone.
2. On Google Play, search for "Interton" and the name of the app
3. After finding the app, tap "Install" then "Accept" to allow the app to access certain features on your phone such as Bluetooth
4. After entering the password, the Interton app will download and install
5. When installation is complete, tap "Open" to open the Interton app

Interton Sound app



The Interton Sound app offers unprecedented user friendliness with just one tap of a finger. Control and customize your hearing aid settings directly from your mobile device, so that you can personalize your hearing experience to match your sound environments.

The Interton Sound app is available on the App Store and Google Play.

To use the app, you need the following devices:


Hearing aids:

- Interton Ready or later released.


Mobile devices:

- iPhone, iPad or iPod touch.
- Supported Android devices.


Please visit our web site for specific compatibility information and an up-to-date list of compatible mobile devices: **www.interton.com/compatibility**

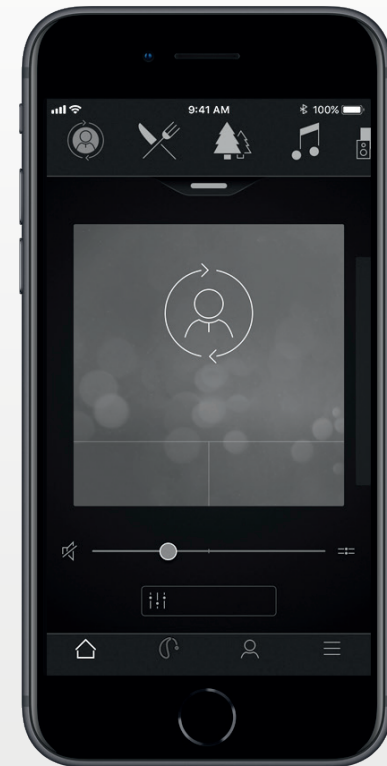


Interton Sound app functionality

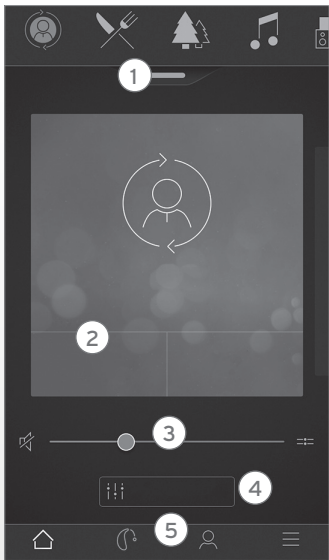

- Main screen:
1. Use the top carousel, **drop-down** or card-swipe to choose your hearing aid or streamer program, or your Favorite. Tap drop-down to edit programs and Favorites.
 2. Use **Quick Buttons** for one-tap, advanced sound adjustments.
 3. Adjust **volume** or mute.
 4. Access **Sound Enhancer** to adjust treble/mid/bass. Also access Tinnitus Manager to adjust pitch and variation of sound from the Generator.
 5. **Navigational menu.**

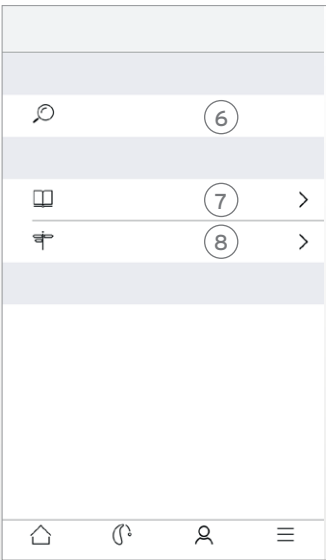

- 'My' menu:
6. Find your **lost hearing aid**.
 7. **Learn about the app** and its functions.
 8. **Guiding tips** for use of the app and the hearing aids.

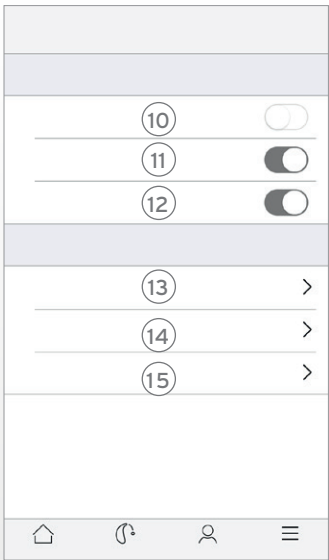

- More menu:
10. Enter/exit **Demo mode**.
 11. Enable/disable **Guiding tips**.
 12. Enable/disable **Automatic activation of Favorite locations**.
 13. **About:** Read about the app and the brand.
 14. **Legal information:** Learn about manufacturer, terms & conditions and privacy policy. Review, give or withdraw consent to data-processing.
 15. **Support:** More app help and additional links to web-based help and FAQs.

Intended use of mobile device apps:

Mobile device apps are intended to be used with wireless hearing aids. Mobile device apps send and receive signals from the wireless hearing aids via mobile devices for which the apps have been developed.



Interton A/S
Lautrupbjerg 7
DK-2750 Ballerup
Denmark

Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to Interton A/S

Interton EasyHearing app



The Interton EasyHearing app provides control of your hearing aids. Used with the Phone Clip 2, Interton EasyHearing lets you use your mobile device to easily obtain an overview of and to remotely control your Interton wireless hearing aids and Interton wireless accessories.

The Interton EasyHearing app is available on the App Store and Google Play.

To use the app, you need the following devices:

Hearing aids:

- Interton wireless hearing aids.

Wireless accessory:


- Interton Phone Clip 2.

Mobile devices:

- iPhone, iPad or iPod touch running iOS 7 or later.
- Android devices* running Android OS 2.3.5 or later.


For best performance, the Interton EasyHearing app should not be used actively on more than one mobile device at a time.

For more information, please consult **www.interton.com**

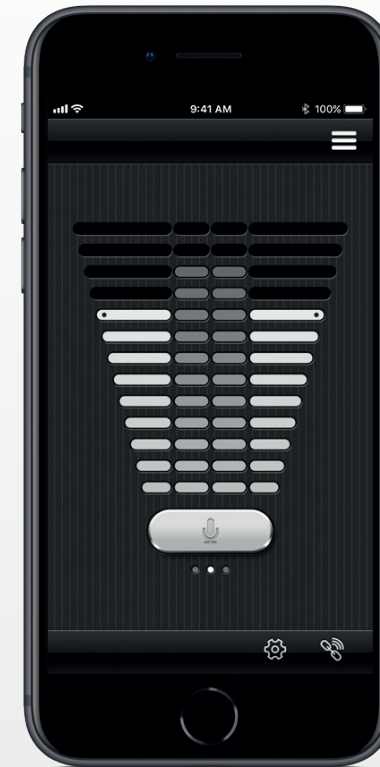


*Functionality verified on selected models from HTC, Huawei, LG, Motorola, Samsung and Sony.

Interton EasyHearing app functionality

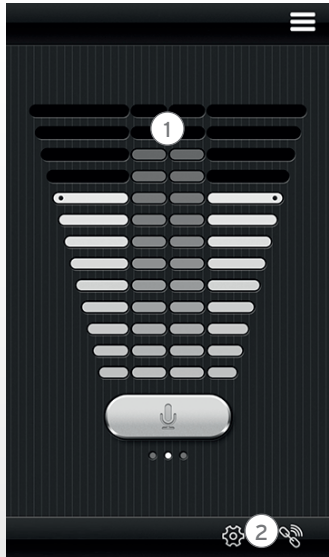

1. **Adjust or mute volume** level on your wireless hearing aids. Adjust or mute volume level on your wireless accessories.

2. Use the dedicated **link screen** to overview connections between the app and your hearing aids.

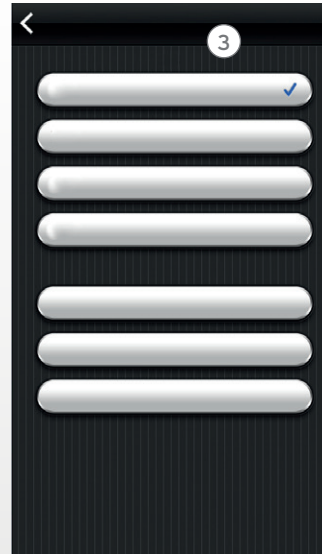

3. **Change manual and streamer programs.**

Intended use of mobile device apps:

Mobile device apps are intended to be used with wireless hearing aids. Mobile device apps send and receive signals from the wireless hearing aids via mobile devices for which the apps have been developed.



Interton A/S
Lautrupbjerg 7
DK-2750 Ballerup
Denmark

Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to Interton A/S

Warnings and precautions

Use with mobile device apps:

Notifications of app updates should not be disabled, and it is recommended that the user installs all updates to app and operating system to ensure highest protection against vulnerabilities.

The app must only be used with Interton devices for which it is intended. Interton takes no responsibility if the app is used with other devices or if third party apps are used to control the Interton devices.


Warnings and cautions

If using the Interton app with the hearing aid, please make sure to read the hearing aid user guide. The hearing aid user guide was included in the hearing aid package. If you need a fresh copy, please consult Interton customer support.

Be aware of information marked with the warning symbol



WARNING points out a situation that could lead to serious injuries,
CAUTION indicates a situation that could lead to minor and moderate injuries.

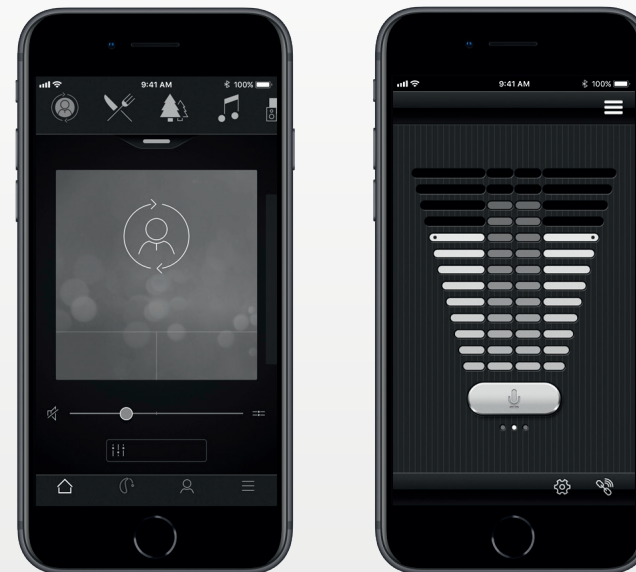



Manufacturer according to
EU Medical Device Directive
93/42/EEC:

Worldwide Headquarters

Interton A/S
Lautrupbjerg 7
DK-2750 Ballerup
Denmark
Tel.: +45 4575 1111
interton.com

CVR no. 55082715