

# Remote Control



## Product description

The Remote Control is for clients who appreciate the ability to easily and discreetly control their hearing aids.

They can use it to change programs and adjust volume, without even looking. This is thanks to a simple keypad that features large buttons, which are concave, tactile and ergonomic. They have raised symbols in the centre, so they are easy to locate and glide a finger over. This is especially good for users who struggle with manual dexterity.

The Remote Control is small and handy and has a streamlined, elegant design. It fits in a handbag, a pocket or the palm of the hand like a keyless car key – and will easily blend in with other smart accessories on a desk or a dinner table.

### Easy to connect

- To pair with a hearing aid, press the pairing button on the back of the Remote Control and open and close the battery door of the hearing aid within 20 seconds.

### Easy to use

- You can change programs in your hearing aid with a simple push on the program button.
- Your hearing aid volume can be adjusted with the + and – button.
- The standard CR2032 Lithium coin cell battery is widely available and easy to replace.

Technical data	
Dimensions	73 x 35 x 14 mm
Weight	22.5 g
Power supply	CR2032 Lithium coin cell battery
Wireless connections	2 hearing aids per Remote Control
Wireless frequency	2.4 GHz
Wireless range	Approximately arm's length distance
Communication speed	2 Mbit/s
Emission	Max. +4 dBm, in compliance with all relevant standards
ESD tolerance	+/- 4 kV contact. +/- 8 kV air.
Operating temperature	0 to +55 °C / +32 to +131 °F
Storage temperature	-20 to +60 °C / -4 to +140 °F

### Worldwide Headquarters

GN Hearing A/S  
Lautrupbjerg 7  
DK-2750 Ballerup  
Denmark  
Tel. +45 45 75 11 11

CVR no. 55082715

© 2019 GN Hearing A/S. All rights reserved.